MANUAL

EN/DK/NO/SE/DE/ES/FR/NL/PL

User Guide

Brugermanual

Bruksanvisning

Bruksanvisning

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ROLLATOR WITH
BRAKES AND A BASKET

thansen

CONGRATULATIONS ON YOUR NEW ROLLATOR

Dear customer,

Thank you for choosing our rollator. This guide contains important information about your rollator and its features.

Please read the manual carefully before using the rollator.

Improper use of the rollator may result in personal injury.

These instructions are part of the rollator and should always be followed. Some illustrations and descriptions in this guide may differ from the model you have. We reserve the right to change technical specifications without prior notice.

Subject to printing errors.

If you have any questions about your rollator, please contact the thansen.dk branch where you bought it or our customer service on +45 31 31 31 31, who are always ready to help.

We hope you enjoy your new rollator.

thansen

INTRODUCTION TO THE ROLLATOR

- Do not start adjusting or using your rollator until you have read these instructions.

ASSEMBLY INSTRUCTIONS

- Take all the parts out of the box.
- Open the frame by pushing down on the hinged folding mechanism until the unit is level.
- Attach the tray, handle and basket
- Put the handles on the frame of the rollator, making sure that the cables are outside the frame
- Loosen the bolts and the height adjustment handle
- Adjust the height of the handles to suit the height of the user. Insert the bolts from the inside so that the head fits into the hole
- Place the spacer and tightening handle on the bolt on the outside of the frame and tighten.
- Hang the basket on the front of the rollator on the hooks provided.

USER GUIDE AND SAFETY INSTRUCTIONS

> USER WEIGHT: MAX 130 KG.

- Make sure the rollator is fully open, the seat is down and the handle under the seat is pushed to the locked position.
- Make sure that all handles and height adjustments are secured and that wheels and other moving parts are in good condition before using the rollator.

- Make sure that both height adjustment handles are tightened and that the handles are at the same height.
- Test the handbrakes before using the rollator. When the brake is applied, the rear wheels should not be able to turn. If the rear wheels are still moving while braking, the rollator must be adjusted. When the brakes are released, the rear wheels must move freely.
- Replace the wheels if they become worn or damaged.
- All wheels must be in contact with the ground when in use. This ensures that the rollator is well balanced.

> TO PREVENT THE ROLLATOR FROM TIPPING OVER:

- Avoid reaching for things that are so far away that you have to move forward on the seat.
- Avoid hanging things on the rollator. Everything should be placed in the basket or the walking stick holder.
- The back support (if there is one) is not designed to support body weight.
- If you need to pick something up, place both feet firmly on the ground and reach down between your knees.
- Do not add anything to your weight when sitting.

The above situations can lead to incorrect weight distribution and cause the rollator to tip over with the risk of injury. Always pay extra attention when reaching for things.

> EN_USER GUIDE

- The brakes must be locked when using the seat.
- Do not use the seat if the surface is uneven.
- Do not sit on the seat while someone else pushes you.
This rollator is intended solely as an aid for walking, and is not a means of transport.

HANDBRAKE INSTRUCTIONS

> ALWAYS USE BOTH HANDBRAKES AT THE SAME TIME.

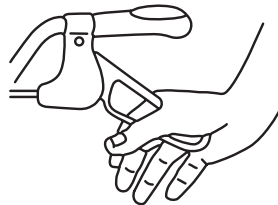
To slow down or stop the rollator temporarily:

- Squeeze the brakes with your fingers until the rollator slows down or stops.


To brake the wheels more permanently:

- Stand between the handles and press the rounded parts downwards with your palm. You will feel and hear a "click" when the wheels are locked.

To release the brake:

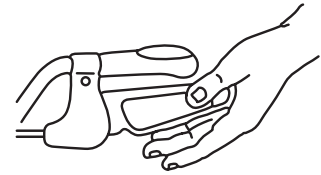- Stand between the handles and push the rounded part up with your palm to the original position.

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